



TOUCH

10/06	15/06	20/06	25/06	30/06	35/06	40/06	50/06
Shaping		Finishing					
Torque: 3.0 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm	
Torque: 2.6 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm		Torque: 2.6 Ncm 300 Rpm	

Excellent Cutting Efficiency
Maximum File Stability

10/06	15/06	20/06	25/06	30/06	35/06	40/06	50/06
Shaping		Finishing					
Torque: 3.0 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm	
Torque: 1.8 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm		Torque: 1.8 Ncm 300 Rpm	

EndoArt Touch rotary files fixed %6 taper allow you to prepare at a standard angle along the canal. The developed special heat treatment technology ensures that the Ni-Ti wire has a much more flexible and durable structure and also turns it into a Gold or Blue color. In addition to its high sharpness efficiency, EndoArt Touch Gold has a higher fracture resistance than standard Ni-Ti files. EndoArt Touch Blue allows pre-bending before entering the canal thanks to its high flexibility efficiency with special control memory and compared to standard Ni-Ti files, it has a much higher fracture resistance. It adapts to the shape of the canal in cases with high slopes.

Touch Kit 15/06-20/06-25/06-30/06-35/06-40/06 19/08-15/04-20/04-20/06-25/06-30/06 19/08-10/06-15/06-20/06-25/06-30/06 19/08-15/06-20/06-25/06-30/06-35/06

Touch Gold Rotary Files

- Rotation Files
- Regular Taper
- Better Cutting Efficiency
- x2 times better Cyclic Fatigue Resistance than conventional Files
- Excellent Strength
- No bounce back
- Respect Original Anatomy of Root Canal
- Great Fracture Resistance
- Lasered Taper and Size information on the shank

Control Memory
Heat Treated Wire

Convex Triangle
Cross - Section



Touch Blue Rotary Files

- Rotation Files
- Regular Taper
- Better Flexibility
- X4 times better Cyclic Fatigue Resistance than conventional Files
- Excellent Strength
- No bounce back
- Respect Original Anatomy of Root Canal
- Great Fracture Resistance
- Lasered Taper and Size information on the shank

Wire : Gold
Length : 21mm, 25mm, 31mm
Packaging : 6 Files

Wire : Blue
Length : 21mm, 25mm, 31mm
Packaging : 6 Files



WhatsApp
Get information!